# AX'S ENDO<sup>®</sup>

04 / 06 / 08

Ref. 60300453-D

Your Endo Specialist™

 **MICRO-MEGA®**  
5-12, rue du Tunnel  
25006 BESANÇON Cedex  
FRANCE

**SAV France**

Tel : +33 (0)3 81 54 42 15  
Fax : +33 (0)3 81 54 42 39  
atelier.reparation@micro-mega.com

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## I. WARNINGS

### I.1 General warnings

- The device is meant for professional dental use only.
- Perform regular maintenance.
- Use the contra-angle only for operations of endodontics. Any other use or alteration of the device is banned and may be dangerous.
- Verify that only rotary instruments in good condition are used.
- Verify that the instrument is properly and firmly held before starting the contra-angle.
- Use a motor in good condition.
- Avoid working with a contra-angle which heats up abnormally.
- It is recommended to wear appropriate protection gloves and eye-protection while handling the contra-angle and the rotary instruments.
- Use a dental dam during intervention on a patient.
- During long treatments, repeat lubrication every ten minutes.
- Never dismantle the contra-angle. Any dismantling of the contra-angle may only be carried out by an approved dealer or the manufacturer.
- Do not activate the push-button while the contra-angle is running.
- Set the speed of the motor in accordance with the utilized contra-angle.
- Contact the MICRO-MEGA® after sales service as soon as a problem with the contra-angle occurs.

- Stop using the contra-angle in case of damage, overheating, irregular operating sound, excessive vibration, excessive run-out of the instrument or inadequate instrument fixation. In order to prevent further damage, contact the MICRO-MEGA® after sales service or your approved MICRO-MEGA® dealer.

### 1.2 Warnings for reprocessing (cleaning – disinfection – lubrication – sterilization)

- Sterilize and lubricate the contra-angle before first use and after every intervention on a patient.
- Remove the rotary instruments (drills, files, burs, etc.) before reprocessing.
- Passing the contra-angle in a basin or an ultrasound system is banned.
- Soaking the contra-angle in a detergent product provided to this effect must not exceed 15 minutes.
- Only use a detergent-disinfectant product specifically intended to the pre-disinfection of dental handpieces and contra-angles which is compatible with aluminium and does not contain soda based products or aldehyde. Do not use any alkaline detergents or solutions (pH > 7).
- Comply with the concentration recommended by the pre-disinfectant manufacturer.
- Respect the immersion time recommended by the pre-disinfectant manufacturer.
- Open the sterilization bag just before use.
- Excessive use of maintenance products (lubricants, cleaning and disinfectant products) may induce the penetration of these products into the inside of the electrical brush motor and affect its functioning. It is imperative to follow the maintenance instructions accompanying each product.
- In case the contra-angle is not used for an extended period of time, clean and lubricate it in accordance with the instructions hereafter and store it in a dry place.

## II. ADVICE FOR USE OF THE PUSH-BUTTONS

Insertion of the instrument (refer to drawing on page 111):

- Make sure that the motor has been stopped.
- Present the canal instrument in its lodging with a slight rotation until you find the flat.
- Push the instrument to the end while pressing the push-button.
- Release the push-button.
- Make sure that the instrument is properly connected and firmly held.

Removal of the instrument (refer to drawing on page 111):

- Make sure that the motor has been stopped.
- Press the push-button and pull the instrument.
- Release the push-button.

## III. REPROCESSING OF THE CONTRA-ANGLE

### 3.1 General information

**(Follow instructions after every use.)**

- The sterilization process does not affect the performance of dental handpieces and contra-angles. The end of the devices' service life is usually determined by damages due to use.
- Immediately after every use, withdraw excessive residues with a disinfecting wipe.

- Do not use products containing acetone, chlorine or bleach as disinfectants! If using disinfecting wipes or jets, follow the instructions of use for these products.
- It is recommended to reprocess the contra-angle directly after every use. If this is not possible, immediately rinse the contra-angle with water to prevent any impurities, blood or saliva from drying or coagulating inside the device.

### 3.2 Manual cleaning prior to sterilization

#### (Follow instructions after every use.)

The immersion of the contra-angle during pre-disinfecting stage (immersion time must not exceed 15 minutes) is authorized in accordance with the following instructions:

- Use a detergent-disinfectant product specifically intended to the pre-disinfection of dental handpieces and contra-angles which does not contain soda based products or aldehyde.
- Comply with the concentration recommended by the pre-disinfectant manufacturer.
- Respect the immersion time recommended by the pre-disinfectant manufacturer.
- After immersion, rinse abundantly to eliminate the liquid/residues mixture contained in the contra-angle.
- Immediately after rinsing, thoroughly dry the inside of the contra-angle using compressed air.
- Before sterilization, lubricate the contra-angle using an oil spray ("500I" by MICRO-MEGA®).

The last three stages of this cleaning process (rinsing, drying and lubrication) may be performed by automatic devices available on the dental market, following the instructions of use of these devices.

**MICRO-MEGA® recommends using X-Cidol® 2 as pre-disinfectant product.**

### 3.3 Automatic cleaning prior to sterilization

#### (Follow instructions after every use.)

In order to eliminate all handling, after each use on a patient place the contra-angle in a device of the product line X-Cid®. These devices clean, disinfect and lubricate your handpieces and contra-angles without previous dismantling and guarantee a rapid bactericide, fungicide and virucide effect, ensuring the destruction of microbial agents such as HIV, HBV and Tuberculosis. The regular use of a device of the product line X-Cid® significantly extends the life span of your handpieces and contra-angles.

### 3.4 Maintenance (Lubrication) (refer to drawing on page 111)

#### (Follow instructions after every use.)

- Remove the root canal instrument.
- Introduce the "500I" spray into the rear end of the contra-angle.
- Press the spray a number of times lasting several seconds until the liquid coming out of the contra-angle is clean.
- Make the contra-angle turn in order to evacuate excess oil before use.
- During long treatments, repeat the lubrication process every ten minutes.

### 3.5 Sterilization

#### (Follow instructions after every use.)

- MICRO-MEGA® handpieces and contra-angles can be sterilized (wrapped in a sterilization bag) in a steam autoclave at 134°C (275°F) to 138°C (280.4°F) and 2.2 bar during 18 minutes.
- Following manual or automatic preparation for sterilization and lubrication, place the contra-angle in a hermetic sterilization bag:
- Individual packaging: The sterilization bag must be of appropriate size for the contra-angle and not be distended.
- Batch processing: The contra-angle may be placed in an instrument box provided to this effect or in a tray generally used for sterilization. Wrap up the trays using appropriate methods.
- Perform sterilization in a sterilizer compliant with the standard EN 13060. Contact corrosion due to humidity may be possible.
- In case you want to sterilize several handpieces or contra-angles in one autoclave cycle, make sure that the autoclave's maximum load is not exceeded (cf. the autoclave's instructions of use).
- Immediately remove the device from the autoclave once the sterilization cycle is accomplished.
- Respect the sterilization bags' duration of storage indicated by the manufacturer.
- Quality and use of the wrapping material must be appropriate for sterilization and comply with the applicable standards.
- Wrap the contra-angle separately in a retractable sterilization bag.
- Open the sterilization bag just before use.

### IV. REPAIRS

For service or repairs, send the complete contra-angle.

Inspections and repairs may only be carried out by an approved dealer or by the manufacturer.

**MICRO-MEGA® recommends annual service for all contra-angles and handpieces.**

### V. GUARANTEE AND AFTER SALES SERVICE

Your MICRO-MEGA® contra-angle is an accurate medical device, produced under maximum quality and test conditions. To guarantee personal follow-up and traceability of its medical devices, MICRO-MEGA® would like you to return the guarantee card, duly filled in, within 10 days of your purchase. This return ensures that the procedure for after sales care of your contra-angle will be instigated so that MICRO-MEGA® can give you full satisfaction with the service you expect and manage your guarantee correctly.

#### You can also register the guarantee for your contra-angle on our website:

[www.micro-mega.com](http://www.micro-mega.com)

MICRO-MEGA® guarantees your contra-angle (except for accessories such as stops or spray clip) for 12 months against any hidden manufacturing defect, excluding faults due to:

- Improper use.
- Inappropriate maintenance or application, not in accordance with our recommendations and instructions for use.
- Accidental damage (fall, impact, etc.).
- Attempted intervention or repair by personnel not approved by MICRO-MEGA®.
- Attempted product modification.

## VI. SPECIFICATIONS

The exterior of AX'S Endo® is made of materials which can be chemically disinfected with aluminium compatible solutions.

	AX'S Endo® 04	AX'S Endo® 06	AX'S Endo® 08
Type	Reduction ratio 100:1	Reduction ratio 75:1	Reduction ratio 50:1
Class in accordance with the Directive 93/42 CEE	IIA	IIA	IIA
Maxi motor speed	40 000 rpm	40 000 rpm	20 000 rpm
Maxi instrument speed	400 rpm	530 rpm	400 rpm
Motor connection standard (coupling)	ISO Standard 3964	ISO Standard 3964	ISO Standard 3964
Instrument fixation	Push-button	Push-button	Push-button
Instrument type in accordance with NF EN ISO 1797-1 (Instrument type)	Type I	Type I	Type I
Instrument diameter (mm) in accordance with NF EN ISO 1797-1	2.35	2.35	2.35

## VII. ACCESSORIES

### 7.1 References of accessories recommended by MICRO-MEGA®

**Ref. 50700008**

500 l Spray supplied without nozzle

**Ref. 50700017**

500 l Spray supplied with 3 nozzles

**Ref.** Please contact us for further information.

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