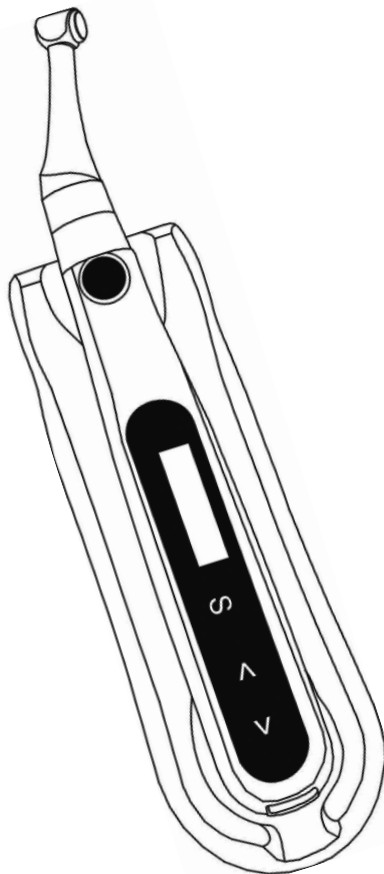
MicroMega

# Dual·Move

**COLTENE**

INSTRUCTIONS

EN

INSTRUCTIONS

FR

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DE

INSTRUCCIONES

ES

INSTRUCÕES

PT

INSTRUCTIES

NL

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RO

KÄYTTÖOHJE

FI

ISTRUZIONI

IT

UTASÍTÁSOK

HU

NÁVOD NA POUŽITIE

SK

BRUKSANVISNING

SV

POKyny

CS

TALİMATLAR

TR

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EL

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JA

# TABLE OF CONTENTS

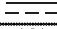
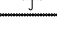
1. Accessories list (Fig. 1) .....	3
2. Symbols .....	3, 246, 247
3. Indication for use .....	3
4. Contraindications .....	3
5. Operating and storage conditions.....	3
6. Charging .....	3
7. Installation of the contra-angle.....	4
8. Installation of the file .....	4
9. Dual Move control .....	4
9. 1. User interface (Fig. 9.A, 9.B, 9.C).....	4
9. 2. Steps to operate .....	4
9. 3. General settings: speed, reverse torque, rotating direction, angles for M0 (Fig. 9.E) .....	5
9. 4. Additional settings (Fig. 9.G).....	6
9. 5. Calibration (Fig. 9.H).....	6
9. 6. Error codes.....	6
10. Dual Pex to Dual Move connection (Fig. 10).....	6
11. Connection testing (Fig. 11).....	7
12. Dual Move screen during treatment (Fig. 12).....	7
13. Combination functions (Fig. 13) .....	7
14. Maintenance.....	8
14. 1. Preamble.....	8
14. 2. Procedure.....	8
15. Main Technical Data .....	11
16. Electromagnetic Compatibility (EMC) – Declaration of conformity.....	11
17. Warranty.....	13
18. Statement .....	13

Please carefully read this manual before use, especially warnings.

## 1. Accessories list (Fig. 1)

<b>1.A</b>	Contra-angle	<b>1.B</b>	Handpiece	<b>1.C</b>	Handpiece base	<b>1.D</b>	Data transfert cable
<b>1.E</b>	Charger	<b>1.F</b>	Spray Nozzle	<b>1.G</b>	Silicon Sleeve		

## 2. Symbols

	Caution		Manufacturer		CE marking		Consult instruction for use
	Product information		Date of manufacture		Direct current		Can be autoclaved up to 134°C
	Serial number		Class II equipment		Comply with WEEE directive		For indoor use only
	Catalogue number		Type B applied part		Keep away from rain		

## 3. Indication for use

Use for dental root canal treatment using endodontic instruments in torque controlled, and in continuous rotation or reciprocating movement. This device must only be used in hospital environments, clinics or dental offices by qualified dental employees and not used in the oxygen-rich environment.

## 4. Contraindications

We recommend not to use the device on patients with heart pacemakers or other electronical implants. The device shall not be used to treat children or pregnant women.

## 5. Operating and storage conditions

- Store and operate the device in a safe and dry environment.
- Do not expose the device to liquids and direct or indirect heat sources.
- To comply with electromagnetic compatibility (EMC) information and avoid disturbance, use the device away from fluorescent lamps, radio transmitters, remote controls or mobile RF communication devices.
- Do not charge, operate, or store the device at high temperatures.
- Gloves and rubber dam are compulsory during treatment.
- If malfunctions occur during the treatment, turn the unit off and contact your after-sales service.
- Never open or repair the device by yourself, otherwise the warranty will not apply.

## 6. Charging

<b>Fig. 6.A</b>	The number of cells shows the battery level. When only one cell, please charge the device.
<b>Fig. 6.B</b>	Connect the adapter to the handpiece as shown.
<b>Fig. 6.C</b>	The screen shows  during charging.

- Keep the device away from any heating source and flammable gas.
- Charge the device when low or no power. Too many charge / discharge cycles may reduce the battery shelf-life.
- Use the specified adapter and battery only. Other models may damage the device.
- Do not use the device while charging.

## 7. Installation of the contra-angle

<b>Fig. 7.A</b>	Make sure the motor is stopped when installing the contra-angle. Connect the contra-angle to the handpiece properly. Use the original contra-angle only.
<b>Fig. 7.B</b>	The contra angle can be 340° rotated without take off, to make it easy to watch the screen during treatment. Put the protective silicon cover onto the contra-angle. When removing after use, pull it straight out slowly.

- Check the proper connection by pulling the contra-angle gently, to avoid unexpected motor reverse during use.

## 8. Installation of the file

<b>Fig. 8.A</b>	To install the file: insert the file and turn it, make sure it is correctly inserted. To remove the file: release the file while pressing the button.
-----------------	--

- Make sure the motor is stopped before installing the file.
- Inspect the file head before inserting it. Do not use the file if its head is damaged.
- Pull the file gently to make sure it is securely connected.

- Insert or remove carefully the file to avoid finger injuries.
- File removal without pressing the button may damage the spindle.

## 9. Dual Move control

### 9. 1. User interface (Fig. 9.A, 9.B, 9.C)

<b>9.A.1</b>	Contra-angle	<b>9.A.4</b>	< / - key to select a memory or other parameters
<b>9.A.2</b>	Main switch (power on, motor start / stop)	<b>9.A.5</b>	> / + key to select a memory or other parameters
<b>9.A.3</b>	Key for selection.	<b>9.A.6</b>	User screen
<b>9.B</b>	<b>Standby screen</b>	<b>9.B.5</b>	Reverse Torque value
<b>9.B.1</b>	Battery level	<b>9.C</b>	<b>Display torque interface</b>
<b>9.B.2</b>	Memory number	<b>9.C.1</b>	Torque scale (N.cm)
<b>9.B.3</b>	Running speed	<b>9.C.2</b>	Cursor of the reverse torque value
<b>9.B.4</b>	Rotation direction	<b>9.C.3</b>	Real time torque display

### 9. 2. Steps to operate

#### 1. Power on / Power off with ● (9.A.2 button)

To power on: push the 9.A.2 button (●), the standby screen is displayed. Automatic switch off if Dual Move is not used for a while (time-out to be set).  
To power off: press simultaneously 9.A.2 (●) and 9.A.4 (⏏) buttons.

#### 2. Select memory program with < / > buttons (Fig. 9.D)

7 different customizable programs. M0: reciprocating mode. M1...M6: continuous rotation

\* M0 Reciprocation mode is only available when specified on the box

- Never use reciprocating files in continuous rotation.
- Never use continuous rotary instruments in reciprocating mode.

**3. Start the rotation: press 9.A.2.**

The real-time torque bar is displayed.

**4. Stop the rotation: press the main switch 9.A.2.**

The standby screen is displayed.

- Make sure the device is functioning properly before using it in mouth.
- Change the file on time to avoid file separation within the canal. File separation may occur due to cyclic / torsional fatigue.
- Heavy force / hand pressure on the endo motor while using it may cause a file separation.
- Do not press the button to release the file while the motor is running, otherwise the file may come off and hurt the patient.
- Electromagnetic noise in surroundings environment may interfere with the device operation and may compromise the automatic control of the device. Always check that the display is not disturbed.

- In case of dysfunction, stop using the device and report the issue to your after-sales service.
- Speed increases the risk of a file separation. Please follow the manufacturer's recommendations regarding the speed and check the motor settings before use.
- Use Ni-Ti or stainless steel-made files only.
- Put a silicon cover onto the contra-angle before each use and especially when combined with Dual Pex.
- Do not forget to remove the file from the contra-angle after use.

**9. 3. General settings: speed, reverse torque, rotating direction, angles for M0 (Fig. 9.E)**

Select the memory (program) with < / > keys then the parameter to be set.

<b>9.E.1</b>	Standby interface continuous mode	<b>9.E.5</b>	Standby interface of M0 mode*
<b>9.E.2</b>	Speed setting	<b>9.E.6</b>	Forward angle setting for M0*
<b>9.E.3</b>	Reverse torque setting	<b>9.E.7</b>	Reverse angle setting for M0*
<b>9.E.4</b>	Direction setting		

- 1 - Touch < / > (- / +) to choose a memory number
- 2 - Touch the S key to select the parameter to be set
- 3 - Touch < / > (- / +) to adjust the parameter

<b>M1 ... M6</b>	Continuous rotation	Rotation speed	120, 150, 200, 250, 300, 350, 400, 450, 500, 550, 600, 650, 700, 800, 900, 1000 rpm
		Rotation direction	Fw: Forward (clockwise rotation) Rev: Reverse (counter clockwise rotation)
		Reverse torque RT	0.5, 0.8, 1.0, 1.2, 1.5, 2.0, 2.5, 3.0, 3.5, 4.0 N.cm
		If the effort applied to the file reaches 70% of RT: discontinuous audible alarm. If the torque exceeds RT: continuous alarm, the auto-reverse is activated to disengage the file. When the torque becomes again less than RT, then the device automatically restarts.	
<b>M0* (op-tional)</b>	Reciprocation*	Forward: 0°, 30°, 50°, 60°, 90°, 150°, 170°, 210°, 250° Reverse: 0°, 30°, 50°, 60°, 90°, 150°, 170°, 210°, 250°	

\*M0 Reciprocation mode is only available when specified on the box

At each change, the new parameters are saved automatically. Automatic switch to the standby screen if no action for a while (time-out to be set, factory value 5 seconds).

- Please set the parameters according to the file manufacturer's recommendations.
- The use of torque reverse function can effectively protect the file from separation within the canal.
- If the auto reverse is too frequent during use, please recapitulate, irrigate and lubricate the root canal or increase the torque as per file manufacturer's recommendations.

**9. 4. Additional settings (Fig. 9.G)**

- 1 - When the device is powered off, press simultaneously the S and the main buttons.
- 2 - Touch S to select the advanced parameter to be set.
- 3 - Touch < / > (- / +) to set values.
- 4 - Press the main switch to escape this mode and come back to the standby screen.

<b>9.G.1</b>	<b>Volume of beep (BEEP VOLUME)</b> Low (Low), middle (Mid) or high (High) volume of audible signals. Factory setting: Middle (Mid).	<b>9.G.4</b>	<b>Inactivity time (SET TIME)</b> Automatic return to the standby display after the specified time of inactivity: 1-15s. Factory setting: 10s.
<b>9.G.2</b>	<b>Auto Power off (AUTO P.W.R)</b> Automatic power off after 3, 5, 10 or 15 min. Factory setting: 10 min.	<b>9.G.5</b>	<b>Calibration (Calibration)</b> Automatic calibration of speed and torque. See next section.
<b>9.G.3</b>	<b>Orientation of the display (Hand)</b> Right (RIGHT) or left (LEFT) orientation. Factory setting: RIGHT Hand.		

When connected to Dual Pex apex locator, Dual Move offers advanced functions, see section 10 to 13.

**9. 5. Calibration (Fig. 9.H)**

Connect the contra angle to the Dual Move handpiece. Do not insert a file.

<b>9.H.1</b>	Enter the calibration interface (see section 9.4) then choose 'ON'. Press the main switch to start the calibration.	<b>9.H.2</b>	Bars show the progress. 2 beeps when completed.
--------------	---	--------------	---

**9. 6. Error codes**

<b>9.I.1</b>	The battery is flat, auto power off will occur. Charge the device immediately.	<b>9.I.3</b>	Long use time, risk of overheating. Stop using the device for a while.
<b>9.I.2</b>	Motor overload. Reduce the load.		

- Follow the instructions of the file manufacturer to set the functions and parameters.
- Calibration is strongly recommended after 10-20 autoclaving cycles for the contra-angle and after its replacement.
- Calibration shall be performed with a 50% battery level as a minimum.
- Do not apply pressure to the contra-angle during calibration.
- If an error code is displayed, please contact your after-sales service.

MicroMega

MicroMega

Dual·Move connected to Dual·Pex (apex locator)

Refer to the Dual Pex user manual to install it correctly before connecting to Dual Move.

**10. Dual Pex to Dual Move connection (Fig. 10)**

- 1 - Make sure that Dual Move is in standby mode.

- 2 - Check that the protective silicon sleeve correctly covers all the metallic surface of the contra-angle.
- 3 - Open the rubber cover and plug the data transfer cable to Dual Move.
- 4 - Power ON Dual Pex and connect the data transfer cable to Dual Pex. CONNECTED ! is displayed on the Dual Move's screen when the connection is correctly done.

## 11. Connection testing (Fig. 11)

<b>Fig. 11.A</b>	Insert a file into the contra-angle, create a short circuit between the lip clip and the file. The file may start rotating if "Auto start / stop" function (section 13, Fig. 13.C). is activated.
<b>Fig. 11.B</b>	Press the Dual Move power switch. A full-length white bar appears next to the line representing the canal, indicating that the system is working properly and is ready to be used. Hang the lip hook on the patient's lower lip and start the treatment.

## 12. Dual Move screen during treatment (Fig. 12)

<b>Fig. 12</b>	The white strip on the Dual Move screen shows the progression of the file into the canal. The beeps frequency increases when the file approaches the apex.
----------------	---

## 13. Combination functions (Fig. 13)

The following advanced functions are only usable when Dual Move is connected to Dual Pex.

- 1 - Touch < / > to select a memory number (M1...M6)
- 2 - Touch the S key for more than one second to enter the advanced function setting interface.
- 3 - Touch < / > to activate (ON) or deactivate (OFF) the advanced functions. If "CONNECT FUNCTION" is deactivated (OFF), all the advanced functions will be inactive, even if they are individually activated (respective status set to "ON").  
The code of following function to be set (APA, ASS, ASD) is displayed at the bottom of the screen, just next to the battery status.
- 4 - Touch S to switch to the following combination function.
- 5 - Touch < / > to activate (ON) or deactivate (OFF) the function then S to go to the next function.

Automatic return to the standby screen if no action for a while (time-out to be set, factory value 5s).

<b>13.A</b>	<b>Connection functions (CONNECT Function)</b> When Dual Move and Dual Pex are connected, set "ON" to have the possibility to use the combination functions, depending on their status.	<b>13.C</b>	<b>Auto Start stop (Auto Stt Stp - ASS)</b> When the file enters the orifice of the canal, the motor will automatically start. When the file leaves the orifice of the canal, it will automatically stop.
<b>13.B</b>	<b>Apical auto-reverse (APA)</b> Automatic reverse when the file reaches the reference point (*). Automatic restart as soon as it comes back above the reference point.	<b>13.D</b>	<b>APICAL Slow Down (ASD)</b> Automatic deceleration once the file exceeds the reference point (*).

Factory status of advanced function for each Memory:

Function	M1, M2, M3	M4, M5, M6
Apical Reverse ( <b>APR</b> )	Activated ( <b>ON</b> )	Deactivated ( <b>OFF</b> )
Auto Start & Stop ( <b>ASS</b> )	Activated ( <b>ON</b> )	Activated ( <b>ON</b> )
Apical Slow Down ( <b>ASD</b> )	Deactivated ( <b>OFF</b> )	Activated ( <b>ON</b> )

(\*) Position of the Reference point (RP) is set through the Dual Pex interface then automatically sent to Dual Move and displayed as a cursor on the Dual Move screen. When the file reaches the reference point, Dual Move will start the APICAL Reverse, APICAL Slow Down depending on their respective status (ON / OFF).

<b>Fig. 13.F</b>	Reference point (RP) defined through the apex locator interface	<b>Fig. 13.G</b>	New position for RP	<b>Fig. 13.H</b>	RP is reached
------------------	---	------------------	---------------------	------------------	---------------

- Use the original data transfer cable only, otherwise the devices may be damaged.
- Carefully manipulate the devices.
- Do not throw, shake or shock the devices. Do not splash devices with liquids.

- Make sure the two devices are connected in the right way.
- After connecting the two devices, gently push and pull the wire plugs to ensure a stable connection and an accurate data transmission.
- In some cases, the determination of the file position may be impossible (blocked canal), or not accurate (abnormal or unusual root canal morphology). An x-ray may be required to confirm results.
- If the displayed file position does not move when the file penetrates the root canal, the device may not work properly. Stop using it and contact your after-sales service.

## 14. Maintenance

### 14. 1.Preamble

- Wear gloves and follow carefully the instructions to avoid cross contamination when performing maintenance.
- Refer to your national guidelines, standards and requirements for cleaning, disinfection and sterilization.

- The device and its components have not been disinfected or sterilized before first use. Please follow the necessary steps before the first treatment.
- Cleaning and disinfection protocols to be applied are specific to each component. Be sure to follow the right procedure below when performing daily maintenance.

### 14. 2.Procedure

- The procedure below shall be followed from step 1 to 6 before the first treatment and after every subsequent treatment.

Step-by-step procedure for autoclavable components.

1 - Cleaning		
Concerned parts	Processing	Warning
Contra-angle PN:51200158  Silicon sleeve PN:51200161	Manual cleaning: Rinse the components off thoroughly under running water for 5 minutes, and brush them for 3 times with a soft brush to remove visible contaminations. Use a soft cloth that has been moistened with mild and aldehyde-free cleaning solution to clean all components for 3 times. Wash the components off in running water for 5 minutes and then wipe off the water with sterile gauze. Use a three-way syringe etc. to blow out any moisture remaining inside the contra angle. Detergent: Metrex EmPower Concentration: 1:128	<ul style="list-style-type: none"> <li>• Remove the file and disconnect the contra-angle before cleaning it.</li> <li>• Do not use an ultra-sonic cleaning device for the contra angle.</li> <li>• If use of washer-disinfectors, use only approved devices according to EN ISO 15883 and refer to their manufacturer's instructions for use. Avoid any contact between the contra-angle and any instrument, kit, support or container.</li> </ul>

Concerned parts	Processing	Warning
	Temperature: 20°C~40°C or Automated cleaning: Carefully put the components into the washer-disinfector on a tray and set the parameters as follows, then start the program: <ul style="list-style-type: none"> <li>• 4 min pre-washing with cold water (&lt;40°C);</li> <li>• emptying</li> <li>• 5 min washing with a mild alkaline cleaner at 55°C;</li> <li>• emptying</li> <li>• 3 min neutralizing with warm water (&gt;40°C);</li> <li>• emptying</li> <li>• 5 min intermediate rinsing with warm water (&gt;40°C);</li> <li>• emptying</li> </ul> The automated cleaning processes have been validated by using 0.5% neodisher MediClean forte (Dr. Weigert).	<ul style="list-style-type: none"> <li>• Make sure the contra angle is dry before moving to the next step.</li> </ul>

## 2 - Disinfection

Concerned parts	Processing	Warning
Contra-angle PN:51200158  Silicon sleeve PN:51200161	Automated Thermal Disinfection in washer/disinfector under consideration of national requirements in regards to A0 value (see EN ISO 15883). A disinfection cycle of 5 min disinfection at 93°C has been validated for the device to achieve an A0 value of 3000. After manual cleaning, the instrument should be automated disinfected or sterilized immediately. A manual disinfection is not recommended.	

## 3 - Drying

Concerned parts	Processing	Warning
Contra-angle PN:51200158  Silicon sleeve PN:51200161	Automated Drying: Drying of outside of instrument through drying cycle of washer/ disinfector. If needed, additional manual drying can be performed through lint free towel. Insufflate cavities of instruments by using sterile compressed air.	

## 4 - Functional Testing, Maintenance:

Concerned parts	Processing	Warning
Contra-angle PN:51200158  Silicon sleeve PN:51200161	Visual inspection for cleanliness of the components and reassembling. Functional testing according to instructions of use. If necessary, perform reprocessing process again until the component is visibly clean. Before packaging and autoclaving, make sure that the device has been maintained acc. to the manufacturer's instruction. Lubrication: Attaching the spray nozzle to oil can and contra angle, press the oil can button more than 3 seconds, till all the black oil flow out from the head of the contra angle.	<ul style="list-style-type: none"> <li>• Before autoclaving, the contra angle must be lubricated.</li> <li>• Only the contra angle needs to be lubricated.</li> </ul>

5 - Packaging		
Concerned parts	Processing	Warning
Contra-angle PN:51200158  Silicon sleeve PN:51200161	Pack the instruments in an appropriate packaging material for sterilization.	<ul style="list-style-type: none"> <li>• Check the validity period of pouch given by the manufacturer to determine the shelf life.</li> <li>• Use pouches which resist to a temperature up to 141°C and compliant with EN ISO 11607.</li> </ul>

6 - Sterilization		
Concerned parts	Processing	Warning
Contra-angle PN:51200158  Silicon sleeve PN:51200161	PSterilization of instruments by applying a fractionated pre-vacuum steam sterilization process (according to EN 285/EN 13060/EN ISO 17665) under consideration of the respective country requirements. Minimum requirements: 3 min at 134 °C (in EU: 5 min at 134°C) Maximum sterilization temperature: 137°C Flash sterilization is not allowed on lumen instruments!	<ul style="list-style-type: none"> <li>• Use only approved autoclave devices according to EN 13060 or EN 285.</li> <li>• The sterilization procedure must comply with ISO 17665.</li> <li>• Waiting for cooling before touching.</li> </ul>

7 - Storage		
Concerned parts	Processing	Warning
Contra-angle PN:51200158  Silicon sleeve PN:51200161	Storage of sterilized instruments in a dry, clean and dust free environment at modest temperatures, refer to label and instructions for use.	<ul style="list-style-type: none"> <li>• Sterility cannot be guaranteed if packaging is open, damaged or wet.</li> <li>• Check the packaging before using (packaging integrity, no humidity and validity period).</li> </ul>

Step-by-step procedure for autoclavable components.

Disinfection		
Concerned parts	Processing	Warning
Handpiece  Handpiece base PN:51200160  Data transfer cable PN:51200159  Adapter PN:51200157	Wipe all the surfaces with a cloth lightly moistened with Ethanol for Disinfection (Ethanol 70 to 80vol%) at least 2min, repeat for 5 times.	<ul style="list-style-type: none"> <li>• Do not use anything except Ethanol for Disinfection (Ethanol 70 to 80 vol%).</li> <li>• Do not use too much ethanol as it's going into machine and damage the components inside.</li> <li>• Do not use any washer-disinfector to clean these components.</li> </ul>

- Use only Ethanol for disinfection (Ethanol 70 to 80% vol.). Do not use too much ethanol to avoid penetration inside the device that may damage internal components.

## 15. Main Technical Data

Dimensions - Weight	20cm x 17.5cm x7cm (package) - 700g
Contra-angle	Compatible with rotary and reciprocating instruments, equipped with Ø2.35 mm nickel titanium root canal file conforming to ISO 1797:2017, Type 1, Files length 11-31mm.
Power supply	Lithium ion battery 3.7V -1500mAh
Charger	AC 100-240V - 50/60Hz - 0.2A
Torque range	0.5 N.cm - 4 N.cm
Speed range	120 - 1000 rpm
Electrical safety class - Applied part	Class II - B
Operating conditions	Indoor use only 10°C / 40 °C - Relative humidity 30% ~75% non-condensing at 0 °C Altitude < 2000 m above sea level
Transport and storage conditions	-20 °C / +55 °C - Relative humidity 20% - 80%, non-condensing at > 40 °C Atmospheric pressure 70 kPa - 106 kPa

## 16. Electromagnetic Compatibility (EMC) – Declaration of conformity

Guidance and manufacturer's declaration			
The medical device is intended for use in the electromagnetic environment specified below. The customer or the user of Dual Move should assure that it is used in such an environment.			
Electromagnetic emission			
Emissions test	Compliance	Electromagnetic environment - Guidance	
RF emissions - CISPR 11	Group 1	Professional healthcare facility environment and Home healthcare environment.	
	Class B		
Harmonic emissions IEC61000-3-2	Class A	Professional healthcare facility environment.	
Voltage fluctuations/flicker emissions - IEC61000-3-3	Complies		
Electromagnetic immunity			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2, 4, 8, 15 kV air	±8 kV contact ±2, 4, 8, 15 kV air	Floors should be in wood, concrete or ceramic tile. If covered by synthetic material, the relative humidity should be at least 30 %.
Electrical fast transients/bursts IEC 61000-4-4	±2 kV 100kHz repetition frequency	±2 kV 100kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	Line to line: ±0.5kV, ±1 kV Line to earth: ±0.5kV, ±1 kV, ±2 kV	Line to line: ±0.5kV, ±1 kV Line to earth: ±0.5kV, ±1 kV, ±2 kV	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply lines IEC 61000-4-11	0% UT; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT; 1 cycle and 70% UT; 25/30 cycles sine phase at 0° 0% UT; 250/300 cycle		Mains power quality should be that of a typical commercial or hospital environment. If the user of devices require continued operation during power mains interruptions, it is recommended that devices be powered from an uninterruptible power supply or a battery.

Rated power frequency magnetic field IEC 61000-4-8	3 A/m 50Hz or 60Hz	3 A/m 50Hz or 60Hz	Power frequency magnetic field should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note: UT: rated voltage(s); E.g. 25/30 cycles means 25 cycles at 50Hz or 30 cycles at 60Hz			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF - IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3V	Portable and mobile RF communications equipment should be used no closer to any part of the medical device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. <b>Recommended separation distance</b> $d=1.2\sqrt{P}$ $d=1.2\sqrt{P}$ 800 MHz ~ 800 MHz $d=2.3\sqrt{P}$ 800 MHz ~ 2.5 GHz
Radiated RF - IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz	3.5 V/m	Where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and $d$ is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.** Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Dual Move is used exceeds the applicable RF compliance level above, the Dual Move should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Dual Move.

\*\* Over the frequency range 150 kHz to 80MHz, field strengths should be less than 3 V/m.

#### Recommended separation distances between portable and mobile RF communications equipment and the medical device.

Nowadays, many RF wireless equipments have been used in various healthcare locations where medical equipment and/or systems are used. When they are used in close proximity to medical equipment and/or systems, the medical equipment and/or systems' basic safety and essential performance may be affected. The Dual Move has been tested with the immunity test level in the below table and meet the related requirements of IEC 60601-1-2:2020. The customer and/or user should help keep a minimum distance between RF wireless communications equipments and the Dual Move as recommended below.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d=1.2\sqrt{P}$	80 MHz to 800 MHz $d=1.2\sqrt{P}$	800 MHz to 2.5 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.  
 NOTE 1: at 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.  
 NOTE 2: these guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## 17. Warranty

Inspections and repairs must only be carried out by Micro-Mega or an approved partner. Your product is an accurate medical device, produced under maximum quality and test conditions. To ensure an individualized follow-up and traceability of your product, please complete the warranty registration on-line on [www.micro-mega.com](http://www.micro-mega.com) website within 10 days of your purchase. This will activate the after-sales procedures for your device so that Micro-Mega can give you full satisfaction. Micro-Mega guarantees the battery for one year, the motor handpiece and contra-angle for two years against any hidden manufacturing defect, excluding faults due to improper use, inappropriate maintenance, accidental damage (fall, impact, etc.), attempted intervention or repairs by personnel not approved by Micro-Mega, attempted product modification.

## 18. Statement

Non-contractual pictures. Micro-Mega reserves the right to modify the characteristics of its product without prior notice.

**/// COLTENE**  
MicroMega

Distributed by Micro-Mega  
12, rue du Tunnel  
25000 Besancon - France

Manufactured by Changzhou Sifary Medical  
Technology Co., Ltd.  
No. 99, Qingyang Road, Xuejia County, Xinbei  
District, Changzhou City, 213000 Jiangsu - China

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Caretechion GmbH